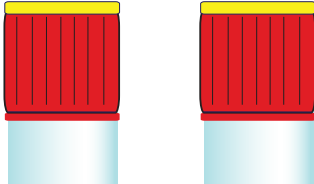
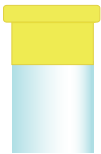
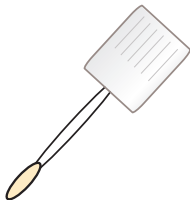


BOSTON HEART DIAGNOSTICS

QUICK REFERENCE SPECIMEN GUIDE

Blood Tubes in Order of Draw	Tests, Alphabetical	
Light Blue Top Tube 9NC coagulation tube with 3.2% sodium citrate 	Fibrinogen	
Red & Yellow Top Tube CAT serum separator clot activator with gel 	Adiponectin Aldosterone Cholesterol Balance (<i>also requires Green Top tube</i>) CoQ10 C-peptide Estradiol Fatty Acid Balance GSP HCY HDL Map (<i>also requires Green Top tube</i>) IGF-1 Insulin	LDL-P & HDL-P by NMR LipoMap™ Test by NMR LpPLA2 NMR Panel OxLDL OxPL-ApoB SARS-CoV-2 Spike IgG Antibody sdLDL PSA, Free TMAO TIBC
Green Top Tube Lithium Heparin plasma separator tube with gel 	Alkaline phosphatase Albumin ALT Amylase Apolipoprotein B Apolipoprotein A-I AST Bilirubin, Direct Bilirubin, Total BUN Calcium Chloride Cholesterol, Total Creatine Kinase (CK) CO2 Cortisol Creatinine Cystatin-C DHEA-S Ferritin Folate Fructosamine FSH GGT Glucose HDL-C hsCRP IL-6 Iron	LDL-C LH Lp(a) Magnesium NT-proBNP Phosphorus Potassium Prealbumin Progesterone Prolactin PTH Thyroglobulin Antibody (TgAb) TPO (thyroid peroxidase antibodies) SHBG Sodium T3, Free T4, Free T3, Total T4, Total Testosterone, Free (calculated) Testosterone, Total Total Protein PSA, Total Triglycerides (TG) Troponin I TSH, Ultrasensitive Uric Acid Vitamin B12 Vitamin D

Blood Tubes in Order of Draw, cont.	Tests, Alphabetical	
White Top Tube K2 EDTA plasma preparation tube with gel 	Myeloperoxidase (MPO)	
Lavender Top Tube K2 EDTA whole blood **Draw 2 lavenders if genetic tests are ordered with CBC and/or HbA1c 	Complete Blood Count (CBC)** HbA1c** Apolipoprotein E Genotype CYP2C19 Genotype Factor II Genotype Factor V Leiden Genotype Haptoglobin Genotype KIF6 Genotype LPA Genotype MTHFR Genotype SLC01B1 Genotype 4q25 Genotype 9p21 Genotype	<i>Saliva preferred, but lavender top acceptable for:</i> Polygenic Risk Map - all profiles

Non-Blood Specimens	Tests, Alphabetical	
Yellow Top Urine Tube Urine transport tube without preservative 	Albumin/Creatinine Ratio, Random Urine	
Saliva Vial Screw cap transport vial with preservatives 	Polygenic Risk Map - all profiles	
Buccal Swab 	<i>Lavender Top (whole blood) preferred, but buccal swab acceptable for:</i> Apolipoprotein E Genotype CYP2C19 Genotype Factor II Genotype Factor V Leiden Genotype KIF6 Genotype	LPA Genotype MTHFR Genotype SLC01B1 Genotype 4q25 Genotype 9p21 Genotype
Stool Sample 	GI Balance	

For more information, call **877.425.1252** or visit **bostonheartdiagnostics.com**.

877.425.1252 bostonheartdiagnostics.com

©2021 Boston Heart Diagnostics Corporation. All rights reserved. The Boston Heart logo is a registered trademark of Boston Heart Diagnostics Corporation in the U.S. and in other countries. Boston Heart Diagnostics Corporation reserves the right to change this document at any time without notice and disclaims liability for editorial, pictorial or typographical errors.
0464_042925

bostonheart
diagnostics®